

CNS VS Local Software Installation

CNS VS4 Local Software is designed to operate/run on any standard laptop and desktop. No special hardware is required. It can be easily deployed at individual practices, busy large group practices, hospital networks, academic medical centers and research projects.

Minimum Computer Configuration:

Windows 7 or greater operating system or Mac OS 10.6.8 or higher

CNS Vital Signs Local Software Applications require:

- **Windows:** Windows 7 or greater operating system. It also requires that the DirectX 'OK' button be selected during the VS4 install. A VS4 Windows install uses about 70MB of disk space. Runs on a 32bit (x86) or 64bit (x64) processor.
- **Apple Mac:** VS4 will run on Mac OS 10.6.8 or higher and using an Intel processor. A VS4 Mac install uses about 50MB of disk space. When you install the Mac version you will be prompted for the Administrator password.

Important: You must install CNS Vital Signs Local Software with a Windows Account that has Administrator privileges. If your computer has multiple accounts setup you will need to log in as the System Administrator or Owner to install the software.

CNS Vital Signs best practices is to Open, Test, Print, Graph, etc. from the Windows USER log-on. Avoid launching the App from both the ADMIN and USER log-ins.


CNS Vital Signs assessment platforms are fully functional when test sessions remain on the counter displayed in the CNS Vital Signs applications initial window. Downloading software upgrades and purchasing additional assessment sessions can be performed by visiting www.CNSVS.com.

To Begin SELECT the 'Download Software' button at CNSVS.com



1

Download and Install the Local Software App

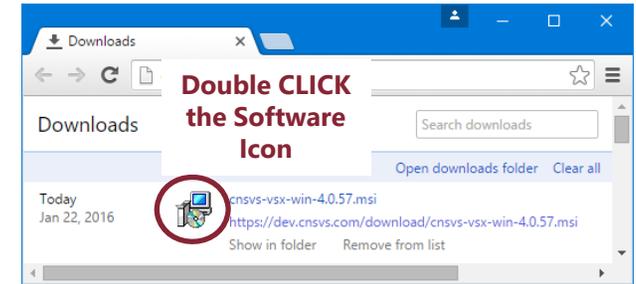


You may encounter a variety of installation warnings depending on the Windows OS you are running. This is from a Windows 10 Installation.


2

You will want to affirm the installation.

Select 'More info'

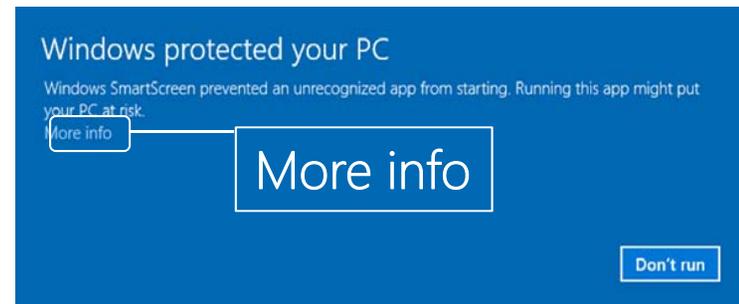

Select Run Anyway

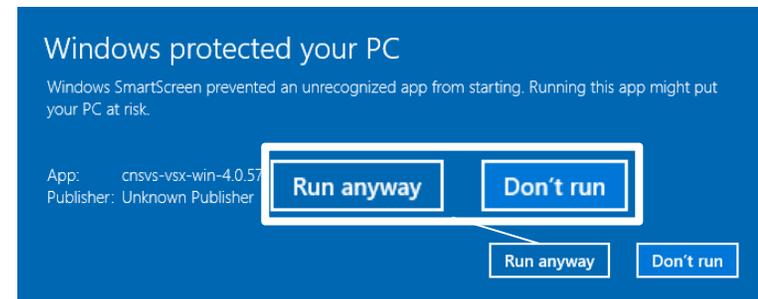

The CNS VS4 installation wizard will appear to begin loading the software.

CNS VS Local Software Installation

3

The installation wizard will appear ready to load the software.

After agreeing to the terms...


SELECT the 'Install' button.



4


You will want to affirm the installation.

You may encounter an installation warning... SELECT Yes, Allow Access, etc.

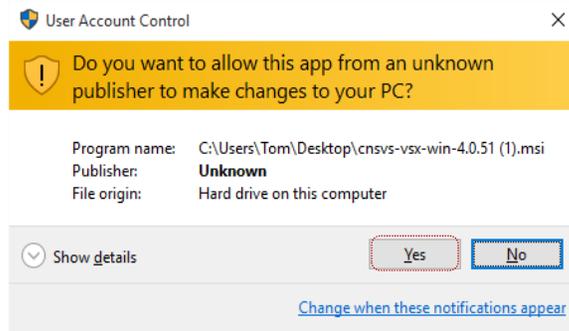


5

The software will install... Continue to select the 'Next' button as needed.

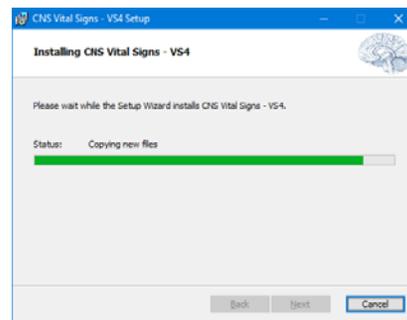

6

SELECT the 'Finish' button

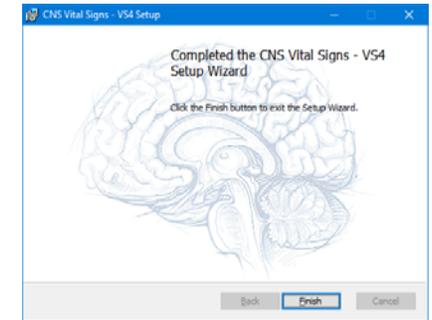


7

Depending on your computers configuration you may be asked to turn on the DirectPlay function

SELECT the 'Install this feature' button

You will be notified of a successful installation

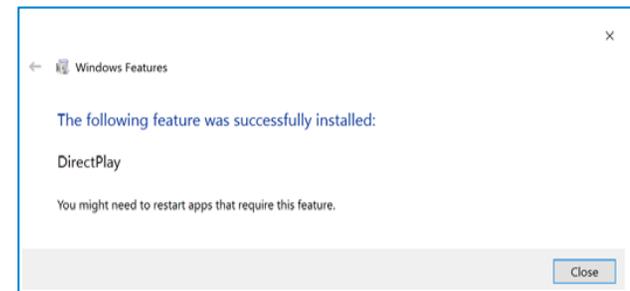

8

Following installation you will be asked to ACTIVATE your software. New Users will receive FREE Training / Trial Assessments when they ACTIVATE their Local Software App.

(See the first page for activation and registration instructions)

