


CNS VS Web – Graph Longitudinal Reports

1 2 3

- ❑ Go to www.cnsvs.com
- ❑ In the upper right-hand corner of the page select **Sign In**
- ❑ Select **View Reports and Manage Account**

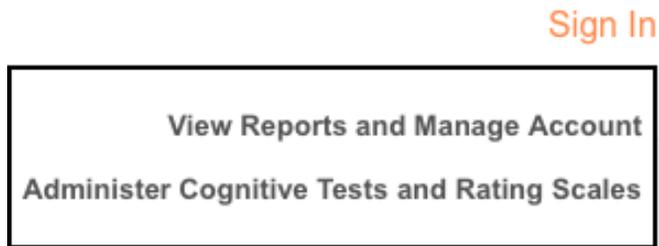

4

- ❑ Select **Graph Longitudinal Reports** in the upper left section of your account management page



5

- ❑ Select a patient ID by checking the correct box in the list, then click **Graph**. The number of test results available for each Patient ID is listed in the Count column. You can select up to 5 Patient IDs to display on one graph. This allows you to include results that were incorrectly identified during testing.

Select up to five Patient ID's...

☑ Add Line to graph Graph Done

Patient ID	Count
<input type="checkbox"/> 295754	2
<input type="checkbox"/> Sample	7
<input type="checkbox"/> SamplePatient	6

... then click "Graph"

Graph Longitudinal Reports